



# austofix EZY-AIM


## ELECTRONIC DISTAL TARGETING

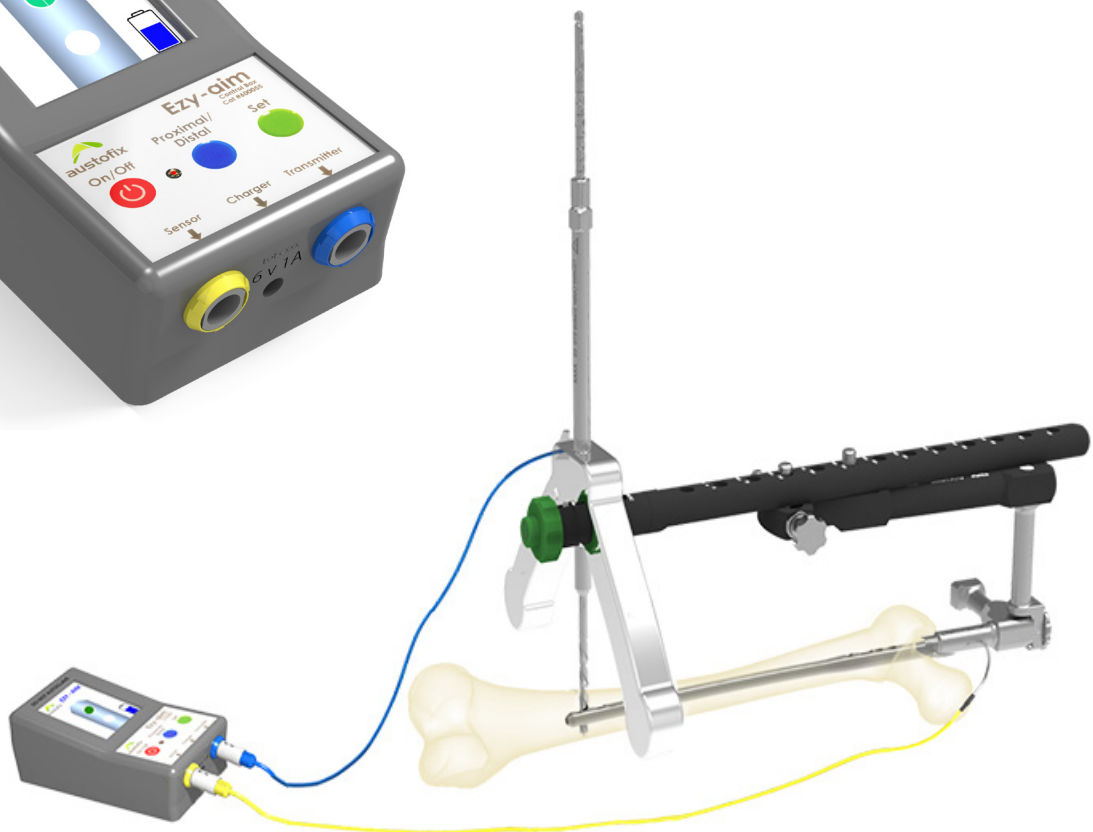

# austofix EZY-AIM

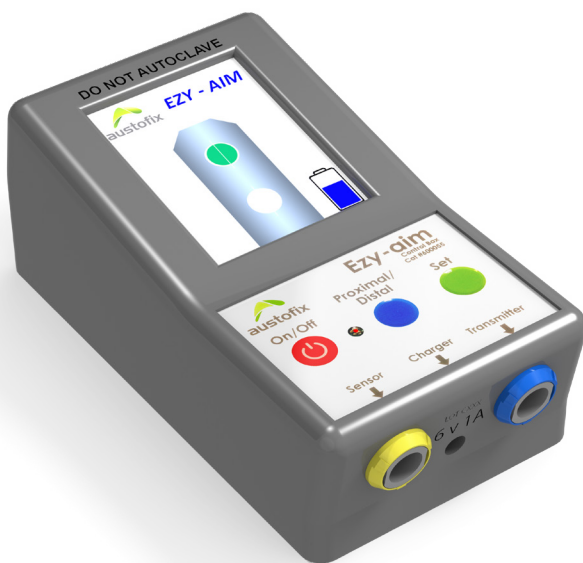


The Austofix Ezy-Aim is an innovative and revolutionary system that embodies precision, reliability and immediacy to provide the surgeon full control during fracture repair in the operating theatre. The system is used in conjunction with the Austofix IM cannulated nails.

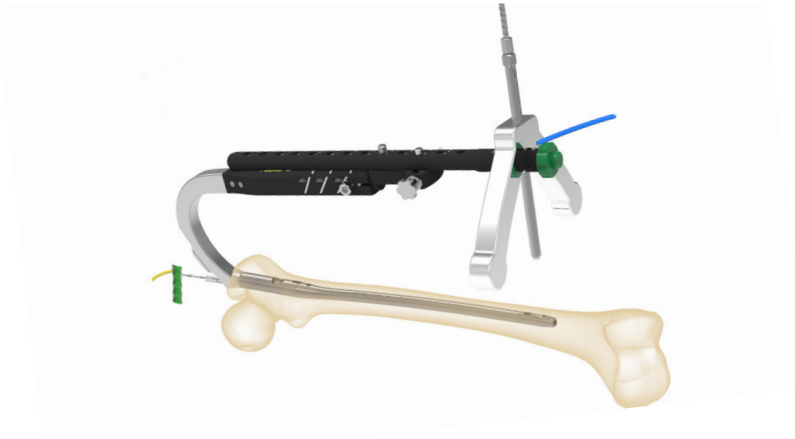

## Features


- Instant set up and ready to use.
- Immediate results with no delay and no x-ray required.
- Real time graphic display feedback.
- All parts are either reusable or autoclaveable.
- Minimal set up and running costs with no consumables.
- Target distal holes independently.
- Rechargeable battery operated and with charge level indicator.

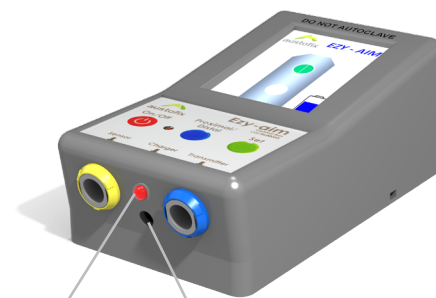

Charging LED

Charging socket


F1 Femoral Nail

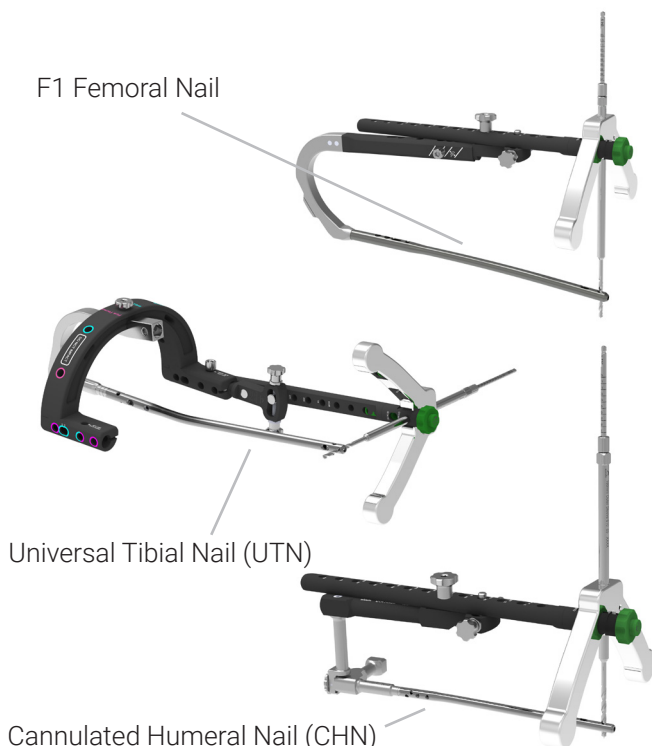

Universal Tibial Nail (UTN)


Cannulated Humeral Nail (CHN)

- Maximum usability with all Austofix cannulated nail sizes.
- Selection of dynamic or static locking for greater flexibility in screw position.
- Pinpoint precision with consistent, reliable accuracy and real time monitoring of the drill site.
- Light, easy to use and compact - size of a notepad.
- Globally cost competitive and superior to all other systems in the market.

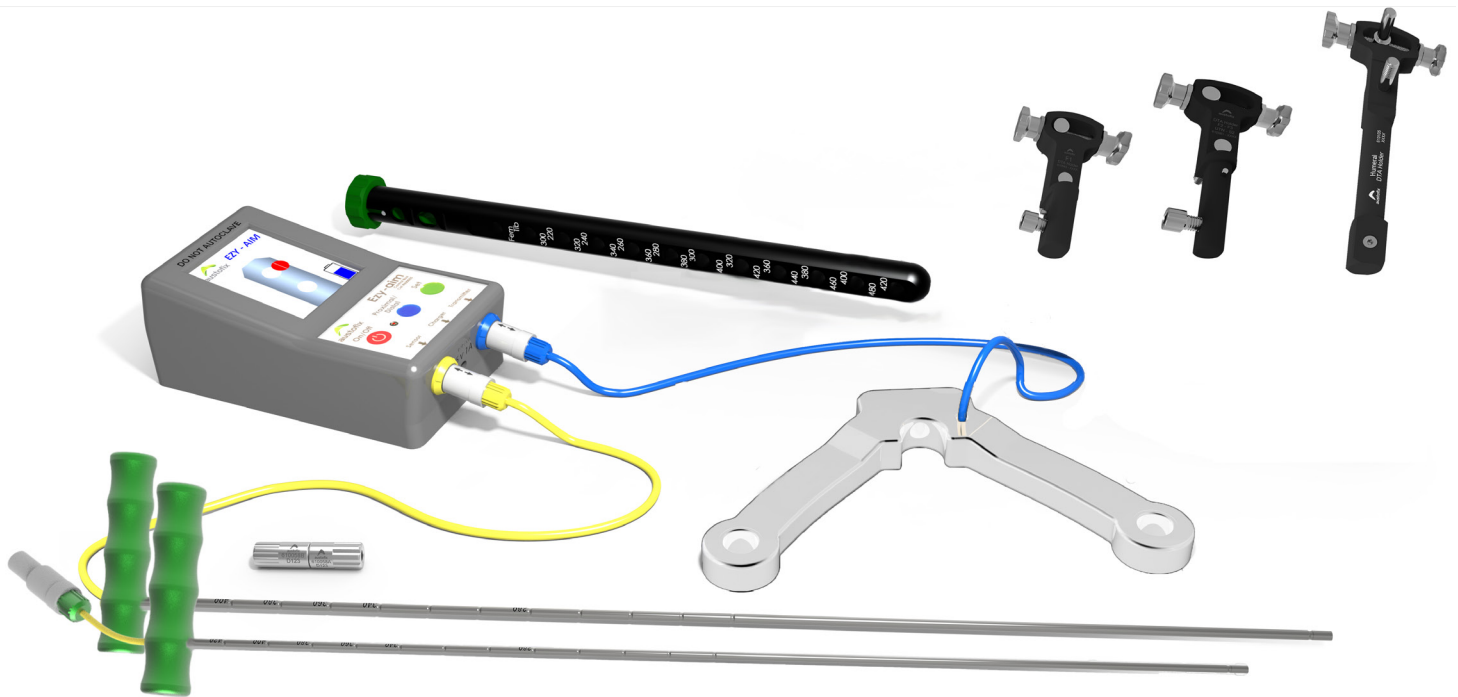

## Ezy Alignment With Ezy-Aim


1. Attach the DTA Holder to the Nail Holder.

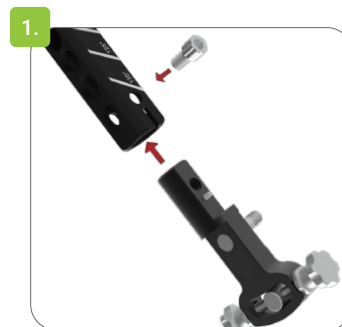

2. Attach the DTA to the DTA Holder.




3. Attach the Transmitter and the Sensor.

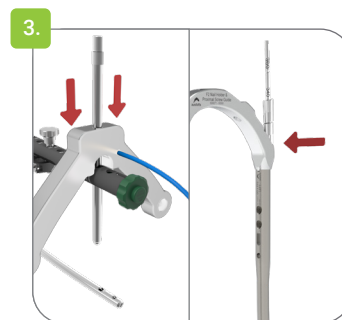

4. Connect the Control Box.



5. Calibrate by using the adjustment knobs to align the drill with distal cross holes.

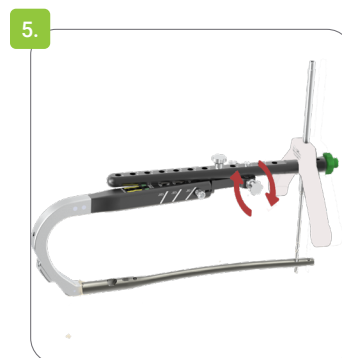


6. Make sure the drill is able to pass through the distal hole. Press 'Set' so that the green dot appears on the display.



The Ezy-Aim system is now calibrated and ready to be used for Nail Targeting.

## Austofix Nails Used With Ezy-Aim System

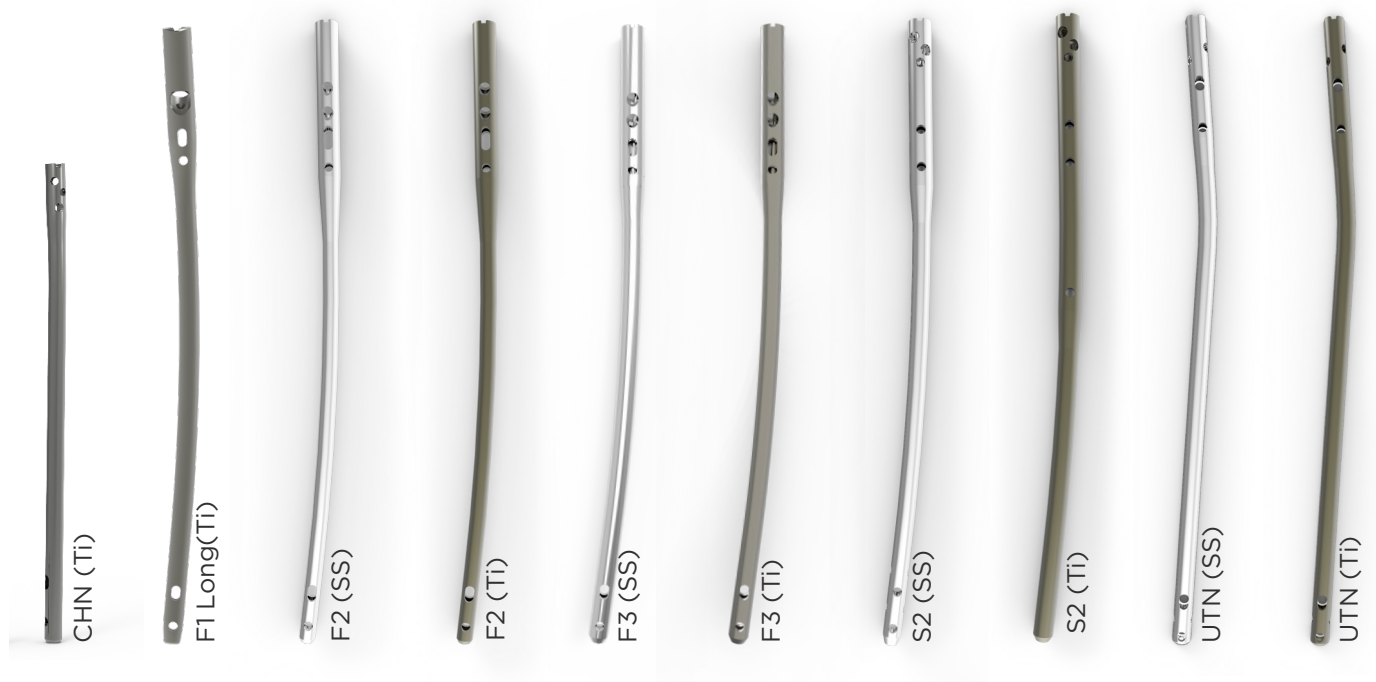

As a leading manufacturer and designer of orthopaedic medical devices Austofix is dedicated to continuous innovation, excellence and patient safety.

Austofix has extensive experience and expertise in developing new medical devices from concept to a fully commercialised product with regulatory approval for worldwide distribution. With a team of world-class research and development specialists Austofix identifies emerging techniques and innovations within the field of orthopaedic to develop world-class solutions.

By focussing on the specific needs of orthopaedic surgeons, Austofix products assist the surgeon in performing accurately, efficiently and safely leading to better health outcomes for the patient. Austofix success can be measured through the excellent clinical results and positive surgeon feedback.

This has led to Austofix emerging as one of Australia's key contributors to the world-wide medical technology industry.



### Manufacturer & Australian Distributor

Australian Orthopaedic Fixations Pty Ltd

Head Office: 28 Dalglish Street, Thebarton South Australia 5031

Registered Address: 18 Kinkaid Avenue, North Plympton South Australia 5037  
AUSTRALIA

CE2797

